



## Instructions for Vapor-Clean Filters

MH Emergency | Suspected Malignant  
Hyperthermia in anaesthetised patient

- 1. Turn off VAPORISER**
2. Do not delay administration of Dantrolene
3. Increase flow rate to >10 L/min
4. Remove circuit, filters, limb, reservoir bag and sample line and dispose
5. Attach Vapor-Clear filters as labelled one to expired and one to inspired
6. Connect new circuit, limb, reservoir bag and sample line
- 7. MACHINE IS NOW MH SAFE, DO NOT REPLACE VAPORISERS**
- 8. TIVA ANAESTHETIC**
9. Maintain fresh gas flow at 10 L/Min for the remainder of procedure

**NB: Vapor-Clear filters are single use.**

**Using this method, Vapor-Clean filters will last 1 hour and MUST be replaced on  
the hour EVERY HOUR**

